

SPEAKING

It is easy to control your speech with only one hand.

NO TOOLS REQUIRED.

Adjustable top head: Twist for fine-tuning of the sound, if needed.
WARNING - over adjusting of the head will result in diminished tone quality; be careful.

On/Off Button: To speak, push the button and release when you are finished.

Push On
Push Off

Fine Tuning



Practice: Place the Romet® R100 Series Electronic Larynx to your neck and practice finding your "sweet spot" to achieve your best vocal tone.

NOTE: This spot can be many places on either your neck or the side of your mouth; each person is different depending upon your skin texture and the amount of radiation you have received. Look into a mirror and repeat simple sentences to achieve vocal clarity. If possible, record your voice and play it back to hear how you sound. Practice regularly!

BATTERY INSERTION



Battery Insertion: Remove the battery end cap from the unit and insert the 9V Lithium battery into the barrel. The polarity of the battery is interchangeable; the + or - battery poles are reversible. Electrical contact will be made automatically when the barrel end cap is screwed onto the end of the barrel.

NOTE: The Romet Model R100 is shipped with one 9V alkaline, non-rechargeable battery (pictured). The 110 Model is shipped with one rechargeable 9V Li-Ion battery and charger. The 120 Model is shipped with two rechargeable 9V Li-Ion batteries and charger.

BATTERY CHARGING

(MODELS 110 and 120 ONLY)




Charger: Included with your Model 110 or 120 unit you will find one (model 110) or two (model 120) 9V Lithium-Ion batteries. When you receive your unit the batteries are only approximately 80% filled to capacity. Charge your battery to full capacity prior to use to achieve maximum power. Plug your Lithium

charger into a 110V circuit (220V will require an adapter for your local area - NOT included in your kit). Insert one or two rechargeable batteries into your charger. Red LED light = Needs Charging. Green LED light = Fully Charged.

WARNING: Charge Li-Ion batteries in only a Li-Ion charger. DO NOT attempt to recharge them with any other charger. DO NOT put any other batteries in the charger, i.e. NiMH. **Normal 9V alkaline batteries will operate in your Romet® electro-larynx.**

ORAL ADAPTER INSTRUCTIONS

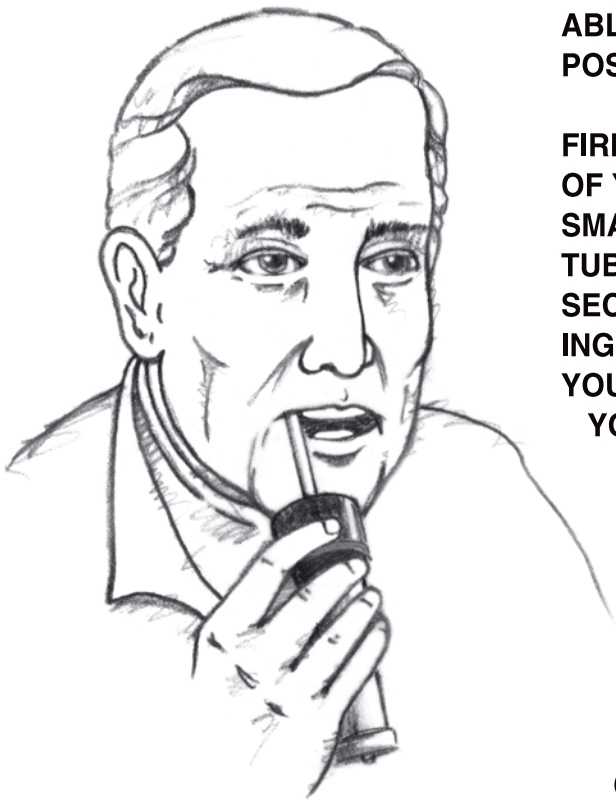

THE ORAL ADAPTER AND TUBES PROVIDED WITH YOUR ROMET ELECTRONIC LARYNX WILL ENABLE YOU TO BE ABLE TO BEGIN SPEAKING WHILE YOUR NECK IS HEALING POST SURGERY.

FIRMLY PLACE THE RUBBER CAP ADAPTER ON THE TOP OF YOUR ELECTRONIC LARYNX. INSERT A TUBE INTO THE SMALL HOLE ON THE TOP OF THE ADAPTER. PUSH THE TUBE FIRMLY INTO THE HOLE. THE TUBE MUST BE SECURE IN THE ADAPTER HOLE WHILE YOU ARE SPEAKING. PLACE THE END OF THE TUBE INTO THE CORNER OF YOUR MOUTH POINTING THE TIP TOWARDS THE TOP OF YOUR MOUTH.

ACTIVATE THE ON-OFF BUTTON. PRACTICE YOUR SPEECH WITH SIMPLE VOWELS OR SHORT SOUNDS. TRY A FEW SIMPLE SENTENCES.

IF POSSIBLE, SPEAK SLOWLY INTO A TAPE RECORDER. REPLAY YOUR SPEECH.

CLEAR TUBES ARE FITTED WITH TIPS TO PREVENT ANY FOOD PARTICLES FROM LODGING INTO THE TUBE WHILE BEING USED. YOU CAN EASILY REMOVE THIS TIP OR SHORTEN THE LENGTH OF THE TUBE WITH SCISSORS.

AFTER FINISHED SPEAKING, RINSE THE TUBES WITH COOL CLEAR WATER. BE SURE TO CLEAN THE TOP OF YOUR ELECTRONIC LARYNX TO REMOVE ANY SALIVA RESIDUE OR FOOD PARTICLES.



SANITAS GRUP Ltd.

1504 Sofia, Bulgaria, 25 "Tzar Osvoboditel" bul., ap. 1
office number: +359 2 943-38-38, office/fax: +359 2 944-16-50